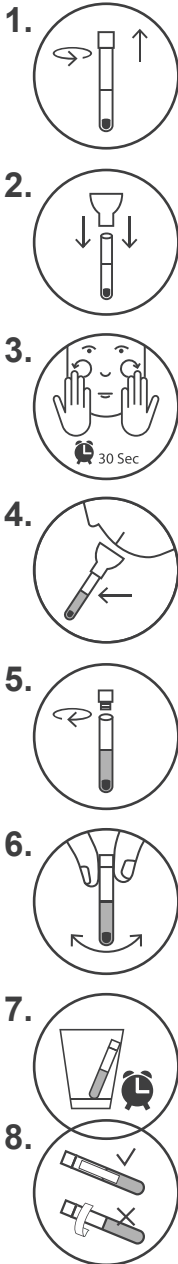
SalivaGene Collector

REF 1035211200

DE 807.01
2025-04-22

DEUTSCH

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SalivaGene Collector Instructions

CE IVD For In Vitro Diagnostic Use*

*) In compliance with REGULATION (EU) 2017/746 on in vitro medical devices
(Not for in vitro diagnostic use in countries where the REGULATION (EU) 2017/746 on in vitro medical devices is not recognized.)

INTENDED USE:

Collection of human saliva samples for DNA stabiliza-tion and extraction (using the SalivaGene DNA Extraction Kits). The DNA in SalivaGene stabilized samples is stable for 1 year at room temperature within the expiry date. The product is intended for use under supervision of professionals only, such as technicians, physicians and biologist trained in molecular biological techniques.

CONTENTS:

Funnel, tube with lid, each tube contains approx.150 mg of SalivaGene reagent.

LIMITATIONS OF THE PROCEDURE:

Do not eat, drink, smoke or chew gum at least 30 min prior to saliva collection. Collecting 2 ml of saliva may require several minutes. Putting some grains of sugar on the tongue helps to produce saliva and does not interfere with the stabilization.

1. Unscrew lid from tube and put it aside for further use.

2. Insert funnel into tube tightly.

3. Rub cheeks against teeth intensely for 30 sec.

4. Collect saliva to indicated fill level, avoid making and measuring air bubbles.

5. Remove and discard the funnel. Screw lid tightly onto the tube again.

6. Shake tube for 15 sec to dissolve white reagent.

7. Store the tube upright for 2-20 min with occasional shaking until Saliva Gene reagent is dissolved. Some cloudy material may occur during this process. This does not interfere with stabilization.

8. For barcode sample tracking stick the small barcode tape vertically onto the tube.

STORAGE:

Store at room temperature 15 – 30°C (59 - 86°F).

SHIPMENT:

Ship according to locally applicable guidelines and regulations such as UN 3373 (Biological Substance, Category B).

SAFETY PRECAUTIONS:

Keep out of the reach of children. Small cap: choking hazard. Do not remove SalivaGene reagent from the tube. Do not use the SalivaGene Collector if content of the tube is spilled.

Do not ingest the contents of the tube. Wash with water if SalivaGene reagent or dissolved SalivaGene

reagent mix comes in contact with skin or eyes.

DANGER:

Contains: ammonium chloride; Hexadecyltrime-thylammonium bromide

Hazard statements

H302: Harmful if swallowed.

H318: Causes serious eye damage.

H410: Very toxic to aquatic life with long lasting effects.

Precautionary statements

P264: Wash hands, forearms and face thoroughly after handling.

P270: Do not eat, drink or smoke when using this product.

P273: Avoid release to the environment.

P280: Wear protective gloves/protective clothing/eye protection/face protection/hearing protection.

P301+P312: IF SWALLOWED: Call a POISON CENTRE or doctor if you feel unwell.

P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P310: Immediately call a POISON CENTER or doctor.

P330: Rinse mouth.

P391: Collect spillage.

P501: Dispose of contents/container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation.

LABEL LEGEND:

LOT	Lot number	UDI	Unique identifier of a medical device
REF	Catalogue number	Expiry date	
	Consult operating instructions	Store at room temperature 15 – 30°C	
	Do not reuse	Amount of sample collectors	
IVD	in vitro diagnostic medical device	CE	CE Marking of Conformity
	Manufacturing date		

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ALS Life Sciences Portugal, S.A.